
Adjusting the BUMPER preferably requires two people.


Remove the straps and position the brace as on the picture below.

The brace must maintain the shoulder and in contact with your chest and your back.


There are three settings for this on each shoulder plate size. Two thicknesses of back pads are also available.

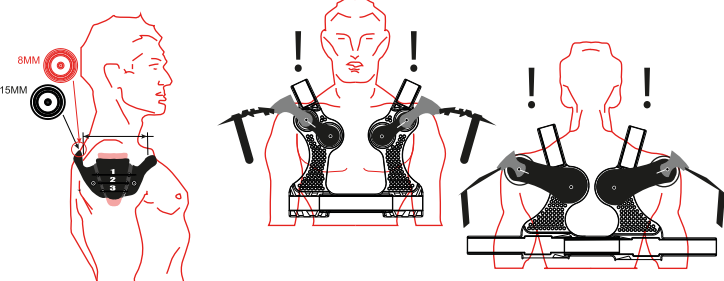Attach the front strap (1) then the back (2) Ensure that the foam is well vertically placed. Attach the side strap (3) and then the top shoulder strap (4) (extension spare part available)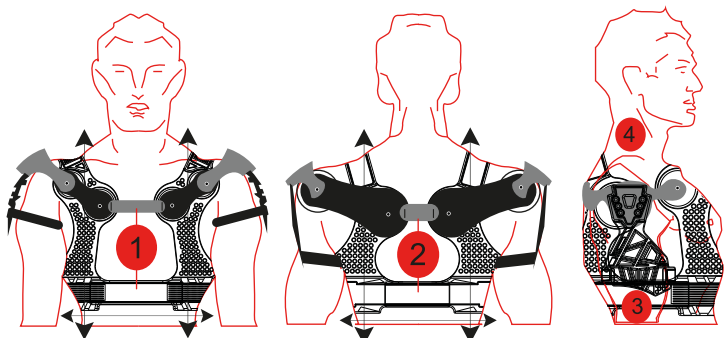

Make sure the brace is positioned like the photo below

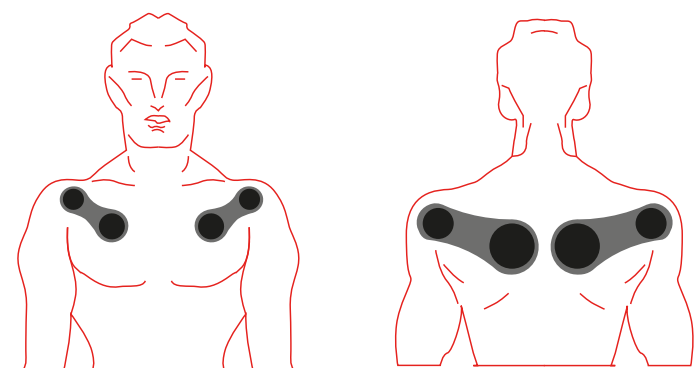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RXR protect Bumper User Guide

Bumper is a shoulder brace (Class 1 medical device EC) designed and made by RXR Protect, 270 Rue de l'Odon, 14790 Mouen, France.

Instructions

Use

Shoulder brace for the shoulder rehabilitation phase, following a dislocation of the shoulder, and in order to avoid recurrent dislocation of the shoulder. An evolutionary support thanks to the adjustment, allowing the rehabilitation of the shoulder.

FEATURES

Semi RIGID PVC frame, articulated

Possible adjustment at each phase, from muscle atrophy to strengthening.

Treatment thanks to the tightening of the different elements, 3 adjustments per plate and two different sizes of plates are delivered with the product.

Pectoral belt, shoulder, upper arm and upper shoulder attachment.

Easy installation.

EVA foam in contact with the body.

Left side and right side inseparable for better support.

Comes with two different sizes of plates to be adapted for rehabilitation

Maintenance

- Store the product in a dry place, away from sunlight and a hot source.
- Hand wash with water. The use of cleaning agents can damage the product and affect its performance.
- Hang the product to dry it.

Hand wash

Inspection before use

- Check the tightness of the screws before each use.

Transport

- The protection must be transported with care to avoid any effort which could damage the product.

Warning

- It is necessary to adjust the product with the user guide to ensure the right positioning of the brace.
- The protection does not completely protect against injuries.
- The user must not modify the product at the risk of affecting its integrity and its performance.

Limitations of use

- Extreme temperatures can affect product performance
- Do not leave the product outside or exposed to environmental elements.

Measurements

The BUMPER product is suitable for bodys above 1m60.

Miscellaneous

- This product contains no harmful substances or hazardous materials
- This product should not be incinerated