XCrossed

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XCrossed

User Guide

X-CROSSED is a knee brace designed and made by RXR Protect, 270 Rue de l'Odon, 14790 Mouen, France.

Instructions

Use

X-CROSSED is a knee brace. It offers fully adjustable knee support thanks to its rigid articulated metal part. Comfortable and ventilated, it avoids twists, hyper flexions and hyper extensions. Suitable for ACL and PCL pathologies

- Designed for knee ligament injuries
- Avoid twists, hyper flexions and hyper extensions

Maintenance

- Store the product in a dry place, away from sunlight and a hot source.
- Hand wash with water. The use of cleaning agents can damage the product and affect its performance.
- Hang the product to dry it.

Hand wash

Inspection before use

- Check the tightness of the screws before each use.
- Check the proper functioning of the articulated mechanism and the correct positioning of the buds.
- After a shock, it is important to inspect the product, make sure that it is in good condition and does not show any damage.

transport

- The protection must be transported with care to avoid any effort which could damage the product.

Warning

- It is necessary to adjust the product with the user guide to ensure the right positioning of the brace.
- The protection does not completely protect against injuries.
- The user must not modify the product at the risk of affecting its integrity and its performance.

Limitations of use

- Extreme temperatures can affect product performance
- Do not leave the product outside or exposed to environmental elements.

Measurements


The X-CROSSED product is suitable for the following sizes:

- Thigh circumference from 38 to 66cm.
- Calf circumference from 28 to 56cm.

Miscellaneous


- This product contains no harmful substances or hazardous materials
- This product should not be incinerated

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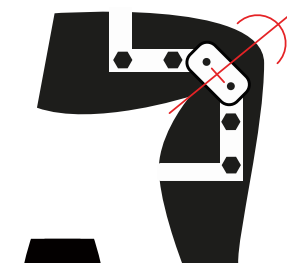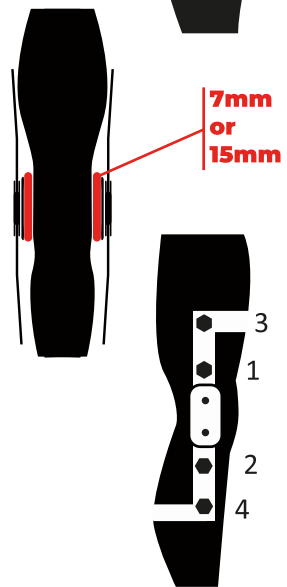
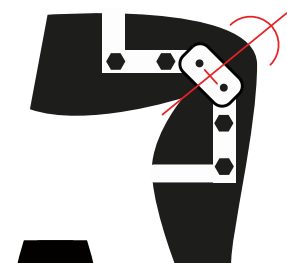
1 : Loosen all the straps of the front and the back part of the brace

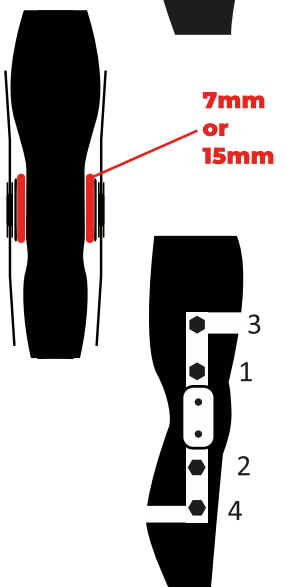2 : Put on the brace
Bend your knee at 45°, place the brace on your leg.

Align the foam pads on the hinged area with the middle of your knee.

3 : Tighten the straps
Start by placing the two straps closest to the knee.

Tightening must allow optimal support of the brace without blocking the circulation of blood.

4: Adjust the range of motion

You can now adjust the range of motion of your orthosis by placing the corresponding

buds in the hole provided for this purpose and then tightening the screw.

- 1: Get up and walk to test your feeling of support and comfort once the brace is placed
- 2: If this is your first use of a brace, it may take a few minutes to get a clear feeling
- 3: The lightening of the Velcro straps should not compromise your blood circulation. Readjust the lightening until you get an optimal feeling of support and comfort.
- 4: The brace does not protect against injuries.